# mamivac®

## ***SENSITIVE-C***

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# mamivac<sup>®</sup>

## ***SENSITIVE-C***

Operating instructions – electric breast pump  
for rental

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**mamivac<sup>®</sup>.com**  
  
**Made in Germany**

Please read the operating instructions carefully before you use the **mamivac® SENSITIVE-C** breast pump and keep them together with the device.

## Contents

|     |                                                 |    |
|-----|-------------------------------------------------|----|
| 1.  | Purpose                                         | 17 |
| 1.1 | Indications                                     | 17 |
| 1.2 | Contraindications                               | 17 |
| 1.3 | Side effects                                    | 17 |
| 1.4 | Suction kits                                    | 17 |
| 2.  | Layout and components                           | 18 |
| 3.  | Using the breast pump                           | 19 |
| 3.1 | Prior to using the pump                         | 19 |
| 3.2 | The mamivac® 2-phase technology                 | 19 |
| 3.3 | The stimulation phase                           | 19 |
| 3.4 | The expressing phase                            | 19 |
| 3.5 | Switching off the device                        | 20 |
| 4.  | Hygiene and cleaning                            | 20 |
| 4.1 | Cleaning and servicing the breast pump          | 20 |
| 4.2 | Cleaning and servicing the mamivac® suction kit | 20 |
| 5.  | Important safety notes                          | 20 |
| 5.1 | Note on electromagnetic compatibility (EMC)     | 21 |
| 6.  | Storage, transportation and ambient conditions  | 21 |
| 7.  | Warranty                                        | 21 |
| 8.  | Practical tips                                  | 22 |
| 8.1 | Storing breast milk                             | 22 |
| 9.  | Troubleshooting                                 | 22 |
| 10. | Environmental protection and disposal           | 23 |
| 11. | Technical data                                  | 23 |
| 12. | Conversion table of units                       | 24 |
| 13. | Legend                                          | 24 |
| 14. | Accessories and spare parts                     | 25 |

## 1. Purpose

With the **mamivac® SENSITIVE-C** breast pump, milk can be expressed and collected gently when breastfeeding problems occur. Breastfeeding problems may be caused by the mother herself or the baby. Using the **mamivac® SENSITIVE-C** electric breast pump, you can:

- increase milk production by completely emptying the breast.
- maintain the breastfeeding option for the baby to continue normal breastfeeding after breastfeeding problems.

**mamivac® SENSITIVE-C** is intended for rental and for home use.

### 1.1 Indications

The **mamivac® SENSITIVE-C** breast pump is used to gently express and collect milk in case of breastfeeding problems or other situations where direct breastfeeding is not possible. Medical indications for using the **mamivac® SENSITIVE-C** breast pump are the following breastfeeding problems:

- mastitis (bacterial inflammation of the mammary glands)
- sore mamillae (nipples)
- anatomic variants of the mamillae (flat or inverted nipples) which make feeding the baby more difficult
- milk oversupply (excessive milk production, poorly sucking child)
- insufficient milk production
- premature babies (poorly sucking premature infant or separation of infant from mother)
- metabolic disorders of the baby
- anatomic disorder in the face of the infant (cleft lip, jaw, palate)
- diseases of the mother where breastfeeding is temporarily contraindicated (e.g. taking medicine).

### 1.2 Contraindications

When taking specific medicine, breastfeeding the infant is contraindicated since pharmacologically effective substances can pass through the milk. For information about possible contraindications for breastfeeding, refer to the Patient Information Leaflet provided by the pharmaceutical company. When medication is limited to a specific period of time, expressing the milk with a breast pump helps to maintain milk production over this period.

**mamivac® pumps may not be used:**

- to suction other body fluids, such as blood, secretions or mucus.
- to suction outside the medical field, such as corrosive, flammable or explosive liquids.
- on other living beings.

### 1.3 Side effects

- In individual cases, skin irritation such as redness or red spots may occur. Less frequently, sore nipples, bruising, or injuries to the nipples are possible.

### 1.4 Suction kits

**For the intended use, you need the stipulated suction kits.**

## 2. Layout and components


- a) Continuously adjustable vacuum control
- b) Continuously adjustable cycle control
- c) Status LED

flashing **green** = ready for use

**yellow** = self-testing

**red** = error message

- d) Connection for power unit
- e) Connection for suction kit
- f) Connection for second suction kit
- g) Sliding cover
- h) Power unit, EU version (24 W) with cable to pump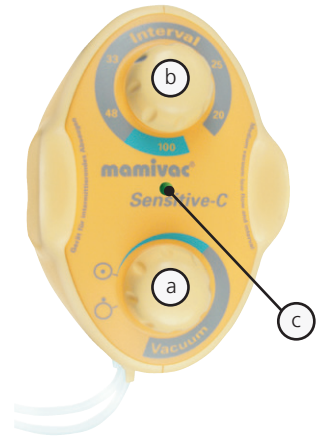


### 3. Using the breast pump

#### 3.1 Prior to using the pump

- Pay special attention to cleanliness. Prior to expressing, always wash your hands thoroughly with soap and wipe the breast with a damp cloth.
- To avoid malfunction and ensure a proper functioning, use the original **mamivac® suction kits** (patient applied part) exclusively.
- Assemble the parts as shown in the operating instructions.
- See Chapter 4 for notes on cleaning and disinfecting.
- Please use air tubes with the following dimensions only:  
5.5 mm outside Ø, 2.5 mm inside Ø, about 1 m long.

#### 3.2 The mamivac® 2-phase technology

The **mamivac® SENSITIVE-C** breast pump simulates the baby's natural sucking behaviour. It can be divided into two phases (stimulation phase and expressing phase).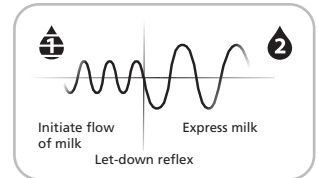


#### 3.3 The stimulation phase

##### 100 cycles; fast cycles, low vacuum

Imitates the baby's sucking which is fast, but weak in the beginning.  
The short cycles stimulate the flow of milk.

- Turn on the pump by switching the vacuum control to . Make sure that the vacuum control indicator is in the area marked green (low vacuum).
- Turn the cycle control to a high cycle number of about 100 cycles (see figure). Make sure that the cycle control indicator is in the area marked green (fast cycles).
- When the milk starts to flow, after two minutes at the latest, switch to the expressing phase, (20-60 cycles; slow cycle, higher vacuum). A beep will sound after 2 minutes to remind you.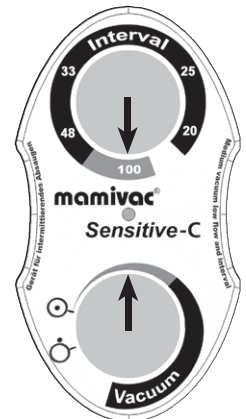


#### 3.4 The expressing phase

##### 20-60 cycles; slow cycles, higher vacuum

When the flow has been stimulated, the milk starts to flow. Now you can express the milk very gently and efficiently.

- Turn the cycle control indicator to the area marked grey (slower cycles).
- Turn the vacuum control indicator to the area marked grey (higher vacuum).
- In the expressing phase, both the cycle speed and the vacuum strength can be continuously adjusted.
- In general, it is recommended to reduce the cycle speed when increasing the vacuum.

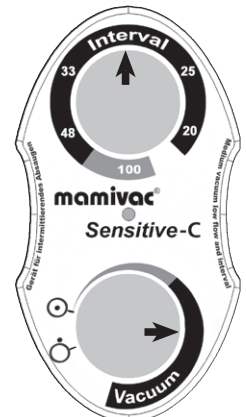### 3.5 Switching off the device

Turn the vacuum control indicator to . The device is now switched off.

## 4. Hygiene and cleaning

### 4.1 Cleaning and servicing the breast pump

- When the pump is not in use or when it is cleaned, always disconnect it from mains supply. In case of problems, also disconnect the pump immediately by all means.
- Do not use aggressive detergents (solvents, etc.) to clean the casing, but a cloth damped with a light soap sud.
- Make sure that no liquid enters the device. Wipe over the parts with a dry cloth. For a disinfection of the casing, you can use alcohol-based disinfectants.

**Do not rinse the pump!**

### 4.2 Cleaning and servicing the mamivac® suction kit

- Do not use chemical disinfection baths, solvents or abrasives.
- All parts of the **mamivac® suction kits**, except for the air tube and the tube connection, can be cleaned by hand with commercial washing-up liquids or in the dish washer or boiled out in a pot at 100 °C for 5 minutes.
- Prior to using the kit the next time, clean the parts which came into contact with milk. With the **mamivac® suction kits**, it is not necessary to clean the air tube or tube connection. It is sufficient to wipe off these parts with a damp cloth. If too much milk enters the bottle, the milk could be sucked up to the protective diaphragm. Switch off the pump immediately. Remove the tube connection immediately from the pump which will release the existing vacuum in the breast shield to prevent damage to the tissue.
- After cleaning, rinse all parts thoroughly with lukewarm water to remove residues of the washing-up liquid, especially on those components which come into direct contact with breast milk. You might use a brush to clean areas which are difficult to reach.
- Please observe the operating instructions of the suction kit.

**Improper cleaning and/or disinfection may lead to contamination with microorganisms and/or their toxins.**

## 5. Important safety notes

- Make sure that the supplied voltage matches the voltage stated on the power unit or is within the limits.
- Do not operate the **mamivac® SENSITIVE-C** breast pump in operating rooms or in rooms with explosion risk. The breast pump is not designed to be used outdoors or during transportation.
- The breast pump is intended for rental.
- Operate and store the breast pump in dry rooms only.
- Operate the **mamivac® SENSITIVE-C** breast pump only on a flat and stable surface.
- In case of disturbances or failure please contact your rental station/vendor.
- If you think a safe operating is no longer ensured, e.g. in case of a faulty insulation of the power cord or faulty cord between the power unit and the device, discontinue the operation of the device immediately (unplug power unit from wall outlet) and secure it against putting into operation again. Unplugging is the only means to safely disconnect the device from the mains supply.

- Any modifications/manipulations of the device or its accessories, or a combination with other devices and/or systems or components make a safe use impossible and are therefore strictly forbidden.
- Small parts are enclosed to the product which could be inhaled or swallowed. Keep them away from toddlers/children.
- Please check the product for any external damage and proper functioning before each use. The pump must not be used again once visible damage has been detected.
- If you experience pain while expressing or are only able to express a small amount of milk, please contact your midwife/lactation consultant or another healthcare professional.
- Any serious incidents that have occurred in connection with the product must be reported to the manufacturer and the competent national authority.

## 5.1 Note on electromagnetic compatibility (EMC)

The **mamivac® SENSITIVE-C** breast pump described in these instructions for use may only be operated in accordance with the following information, provided that the accessories and spare parts mentioned in chapters 11 and 14 are used, in an electromagnetic environment that corresponds to a domestic environment (dwelling, medical practices) where the public power system is available.

It is not permitted to operate the **mamivac® SENSITIVE-C** breast pump in means of transportation, be it over land, across water or through the air, nor in special-use HF shielded rooms (e.g. MRT) where electromagnetic bursts of high intensity may occur nor in the immediate vicinity of active HF surgery devices or shortwave therapy devices.

A detailed technical description of the electromagnetic environment can be found in the enclosed EMC guidelines.

Provided that the **mamivac® SENSITIVE-C** breast pump is operated in the specified environment, an impairment of the performance characteristics given in chapter 11 or the basic safety is not to be expected.

**WARNING:** Avoid using this device directly next to other devices or with other devices stapled on top of each other as this could disturb operation.

**WARNING:** The use of accessories, converters or cables other than those defined or provided by Kimetec for this device can lead to an increased electromagnetic interference or a reduced electromagnetic immunity of the device resulting in faulty operation.

**WARNING:** Portable HF communication devices (transmitters), including their accessories such as antenna cables and external antennas, shall not be used within a distance shorter than 12 inches (30 cm) to the parts and cables listed by Kimetec for the **mamivac® SENSITIVE-C** breast pump. Non-observance may impair the performance characteristics of the device.

## 6. Storage, transportation and ambient conditions

- Avoid strokes or impacts of any kind, regardless of whether the device is packed or not.
- Avoid rapid temperature changes, high air humidity or a dusty environment.
- The device can be operated at an ambient temperature of +5°C to +40°C, a relative humidity of 15 % to 93 % (non-condensing) and an air pressure of 700 hPa to 1060 hPa.
- Store and transport the device at an ambient temperature of -29°C to +60°C, a relative humidity of max. 85 % and an air pressure of 500 hPa to 1060 hPa.
- An excessive accumulation of lint and dust in the inside of the device impairs heat dissipation of components, which may overheat as a result representing a fire hazard.

## 7. Warranty

The warranty period for the **mamivac® SENSITIVE-C** breast pump is 36 months from the date of purchase.

Improper use, opening the device or using suction kits not specified for this breast pump will void any warranty claims.

## 8. Practical tips

Prior to using the pump for the first time and after every usage, clean your **mamivac® suction kits** as described in the respective operating instructions. Please check whether all parts are undamaged. Make sure to assemble the parts properly again as shown in the instructions. Prior to expressing, always wash your hands thoroughly with soap and wipe the breast with a damp cloth. To avoid dry, sore or cracked nipples, do not use soap or alcohol as detergent. Prepare yourself for the expressing with ease. Take a comfortable and relaxed seat.

- Gently press the breast shield over the nipple for a tight contact.
- You can operate the **mamivac® SENSITIVE-C** breast pump either with one or two suction kits. To save time, it is possible to express both breasts simultaneously (double pumping).
- Start to express milk. Please observe the instructions for operating explained in Chapter 3.
- The pump features a stimulation mode to support milk production. You can also use your free hand for an additional breast massage to support the flow of milk.
- Express both breasts alternately, unless instructed otherwise by your physician.
- When expressing make sure not to overfill the milk bottle. With a capacity of 125 ml, we recommend to interrupt expressing when the 100 ml mark is reached and fill the expressed breast milk to another container/bottle. Turn the bottle counterclockwise to unscrew it from the breast shield.
- For further information regarding breast shield size please ask your lactation consultant or midwife.
- Since expressing times strongly depend on the individual mother, you are asked to contact your midwife or breastfeeding assistant. As a rule of thumb, the following expressing times are recommended:

### **With one suction kit:**

Expressing scheme: Express first and second breast 5 minutes each one after the other, followed by 3 minutes and 2 minutes each.

### **With two suction kits:**

Express both breasts simultaneously for 10 minutes (double pumping). Have a look at the fill level of the bottle at the suction kit from time to time and make sure it does not overflow.

## 8.1 Storing breast milk

### **Breast milk can be stored as follows:**

- Freshly expressed milk at room temperature (16°-29°C) 3-4 hours
- In the rear of the fridge (below 4°C) 3 days
- In the freezer (-18°C) 6 months

Fill the used bottles or containers to two thirds only and write down the fill date. Instead of a bottle, you can also use our specially designed **mamivac® breast milk freezer bags** [REF 282100]. Do not defrost frozen breast milk in the microwave, but at room temperature. Place the bottle or bag in warm water as required. After defrosting, gently shake the container or bag to mix the milk.

## 9. Troubleshooting

The hygienic diaphragm of the suction kit effectively prevents that liquids or solids are sucked into the pump. If substances are sucked in nevertheless or any other problem occurs, immediately put the device out of operation. Contact your rental station or directly the manufacturer's customer service.

| Problem:                                      | Error / solution                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
|-----------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Pump does not start                           | <ul style="list-style-type: none"> <li>• Check connections between power supply and device</li> <li>• When mains operated, the LED (c) must light up in green</li> <li>• If you cannot solve the problem, return the device.</li> </ul>                                                                                                                                                                                                                                                                                                        |
| Pump does not suck (does not build up vacuum) | <ul style="list-style-type: none"> <li>• Status LED (c) lights up in red (error message)</li> <li>• Try to switch off and on the device several times. Wait a minute before switching on again. After five unsuccessful attempts, return the device to the manufacturer.</li> <li>• Check the suction kit:</li> <li>• Check breast shield valve (for cracks and soiling)</li> <li>• Check protective diaphragm for cracks</li> <li>• Protective diaphragm must be inserted flush and crinkle free</li> <li>• Check tube connections</li> </ul> |

The device is maintenance-free. For repair work, it must be sent to the manufacturer. The intended function can be checked by the manufacturer or by means of the test unit for mamivac® breast pumps (REF 2899120). Regular testing of electrical safety (recommended at intervals of 2 years) can also be performed by the manufacturer or another authorized person who has the required expertise and measuring and testing equipment.

The breast pump/power unit must not be opened by users/patients.

## 10. Environmental protection and disposal

Please dispose of **mamivac® SENSITIVE-C** breast pumps in accordance with your local disposal regulations for electrical devices.

## 11. Technical data

The expected service life is 5 years.

### Pump mamivac® SENSITIVE-C:

|                    |                                                         |
|--------------------|---------------------------------------------------------|
| Nominal voltage:   | 12 V DC                                                 |
| Power consumption: | 1.25 A                                                  |
| H x W x D (mm):    | 190 x 112 x 145                                         |
| Weight:            | 790 g                                                   |
| Max. vacuum:       | -33 kPa (±15 %)                                         |
| Mean vacuum:       | Low flow intermittent sucking without continuous vacuum |

### REF 281012

### Device classification:

Classified acc. to 93/42/EEC,  
Appendix IX, Rule 11: Class IIa

### Power unit (24 W):

|                                                      |                         |
|------------------------------------------------------|-------------------------|
| (exchangeable plug-in adapters available on request) |                         |
| Nominal input voltage:                               | 100-240 V AC 50 / 60 Hz |
| Input current:                                       | 0,6 A                   |
| Output voltage:                                      | 12 V DC                 |
| Output current:                                      | 2,08 A                  |
| Protection class:                                    | II                      |
| H x W x D (mm):                                      | 110 x 40 x 80           |
| Weight:                                              | 186 g                   |

### REF 283008 EU version

|                                                      |                              |
|------------------------------------------------------|------------------------------|
| <b>Power unit (24 W):</b>                            | <b>REF 283022 EU version</b> |
| (exchangeable plug-in adapters available on request) |                              |
| Nominal input voltage:                               | 100-240 V AC 50 / 60 Hz      |
| Input current:                                       | 490 mA                       |
| Output voltage:                                      | 12 V DC                      |
| Output current:                                      | 2000 mA                      |
| Protection class:                                    | II                           |
| H x W x D (mm):                                      | 95 x 42 x 67                 |
| Weight:                                              | 160 g                        |

## 12. Conversion table of units

|                          |              |                               |
|--------------------------|--------------|-------------------------------|
| <b>1 kPa</b>             | corresponds: | 0.01 bar                      |
| <b>0.14504 PSI</b>       | corresponds: | 7.5006 mmHg                   |
| (pounds per square inch) |              | (millimetre mercury, at 0 °C) |

## 13. Legend

|                                                                                    |                                                             |                                                                                   |                                                                                                          |
|------------------------------------------------------------------------------------|-------------------------------------------------------------|-----------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------|
|    | Observe safety instructions                                 |  | Manufacturer                                                                                             |
|    | Observe operating instructions                              |  | Protect from sunlight                                                                                    |
|    | Store in dry place                                          | <b>IP41</b>                                                                       | Protection against penetration of solid particles of ≥1.0 mm diameter and vertically falling water drops |
|    | Conformity mark according to EU Council Directive 93/42/EEC |  | Direct current                                                                                           |
|    | Device of protection class II                               |  | Alternating current                                                                                      |
|    | Patient applied part of type BF                             |  | Do not dispose in domestic waste                                                                         |
|   | On                                                          | <b>SN</b>                                                                         | Serial number                                                                                            |
|  | Off                                                         | <b>REF</b>                                                                        | Item number                                                                                              |
|  | Observe operating instructions                              |                                                                                   |                                                                                                          |

## 14. Accessories and spare parts

| Designation                                      | REF        |
|--------------------------------------------------|------------|
| mamivac® SENSITIVE-C breast pump (device)        | 281012     |
| mamivac® suction kit PREMIUM L                   | 283120     |
| mamivac® double kit PREMIUM L                    | 283123     |
| mamivac® suction kit PREMIUM XL                  | 283133     |
| mamivac® suction kit PREMIUM S                   | 283129     |
| Power unit for SENSITIVE-C EU (110 x 40 x 80 mm) | 283008     |
| Power unit for SENSITIVE-C EU (95 x 42 x 67 mm)  | 283022     |
| Power unit for SENSITIVE-C UK                    | 283008K01  |
| Power unit for SENSITIVE-C US                    | 283008K02  |
| Power unit for SENSITIVE-C AU                    | 283008K03  |
| Operating instructions mamivac® SENSITIVE-C      | GB1010002M |
| mamivac® suction kit ECONOMY L, small bottle     | 283122     |
| mamivac® double kit ECONOMY L, small bottle      | 283124     |
| mamivac® carrying case SENSITIVE-C               | 283011     |
| mamivac® carrying bag SENSITIVE-C                | 283010     |
| mamivac® Test unit                               | 2899120    |

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