## ***SENSITIVE-CFH***

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# mamivac®

## ***SENSITIVE-CFH***

Operating instructions – electric breast pump  
only for clinical use

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mamivac®.com  
  
Made in Germany

Please read the operating instructions carefully before you use the **mamivac® SENSITIVE-CFH** breast pump and keep them together with the device.

## Contents

1.	Purpose	17
1.1	Indications	17
1.2	Contraindications	17
1.3	Side effects	17
1.4	Suction kits	17
2.	Layout and components	15
3.	Available accessories	19
3.1	Mobile stand	19
3.2	Components of the mobile stand	19
3.3	Assembling	19
4.	Using the breast pump	20
4.1	The mamivac® 2-phase technology	20
4.2	The stimulation phase	20
4.3	The expressing phase	21
4.4	Switching off the device	21
5.	Hygiene and cleaning	21
5.1	Cleaning and servicing the breast pump	21
5.2	Cleaning and servicing the mamivac® suction kit	21
6.	Important safety notes	21
6.1	Note on electromagnetic compatibility (EMC)	22
7.	Storage, transportation and ambient conditions	22
8.	Warranty	23
9.	Practical tips	23
10.	Troubleshooting	23
11.	Environmental protection and disposal	24
12.	Technical data	24
13.	Conversion table of units	24
14.	Legend	24
15.	Accessories and spare parts	25

# 1. Purpose

With the **mamivac® SENSITIVE-CFH** breast pump, milk can be expressed and collected gently when breastfeeding problems occur. Breastfeeding problems may be caused by the mother herself or the baby. Using the **mamivac® SENSITIVE-CFH** electric breast pump, you can:

- increase milk production by completely emptying the breast.
- maintain the breastfeeding option for the baby to continue normal breastfeeding after breastfeeding problems.
- This breast pump is for clinical use only. Not intended for home care.

## 1.1 Indications

The **mamivac® SENSITIVE-CFH** breast pump is used to gently express and collect milk in case of breastfeeding problems or other situations where direct breastfeeding is not possible. Medical indications for using the **mamivac® SENSITIVE-CFH** breast pump are the following breastfeeding problems:

- mastitis (bacterial inflammation of the mammary glands)
- anatomic variants of the mamillae (flat or retracted nipples) which make latching and feeding the baby more difficult
- insufficient milk production
- sore mamilla (nipple)
- milk oversupply (excessive milk production, poorly sucking child)
- premature babies (poorly sucking premature infant or separation of infant from mother)
- metabolic disorders of the baby
- anatomic disorder in the face of the infant (cleft lip, jaw, palate)
- diseases of the mother where breastfeeding the baby is contraindicated over a limited period of time (e.g. taking medicine)

## 1.2 Contraindications

A possible contraindication is being allergic to the silicone of the soft insert which is in direct contact with the skin. When taking specific medicine, breastfeeding the infant is also contraindicated since pharmacologically effective substances can pass through the milk. For information about possible contraindications for breastfeeding, refer to the Patient Information Leaflet provided by the pharmaceutical company. When medication is limited to a specific period of time, expressing the milk with a breast pump helps to maintain milk production over this period.

**mamivac® pumps may not be used:**

- to suction other body fluids, such as blood, secretions or mucus
- to suction outside the medical field, such as corrosive, flammable or explosive liquids.
- on other living things

## 1.3 Side effects

- In individual cases, skin irritation such as redness or red spots may occur. Less frequently, sore nipples, bruising, or injuries to the nipples are possible.

## 1.4 Suction kits




For the intended use of the breast pump you need the stipulated original **mamivac® suction kits** (see chapter 15). The use of other suction kits represents a significant safety risk. Please connect **mamivac® suction kits** to connectors f and/or g). Please see operating instructions for suction kits.

## 2. Layout and components

mamivac® SENSITIVE-CFH as table unit

- a) Continuously adjustable vacuum control
- b) Continuously adjustable cycle control
- c) Status LED
  - flashing green** = ready for use
  - yellow** = self-testing
  - red** = error message
- d) Bottle holder
- e) LED for mains operation
  - green** = ready for use
- f) Connection for suction kit
- g) Connection for suction kit
- h) Connection for power cord
- i) Power cord 3 m (REF 283003)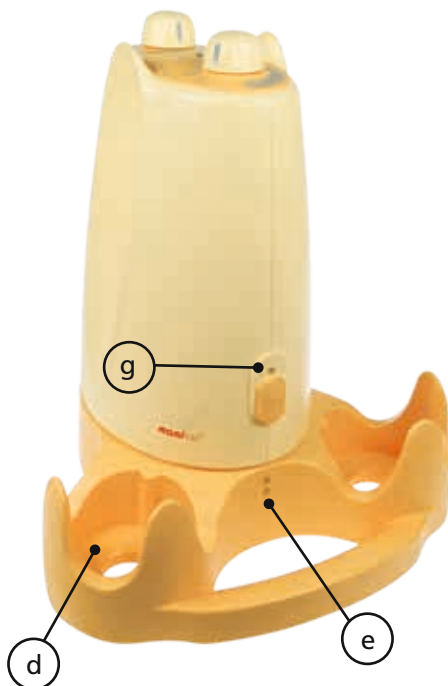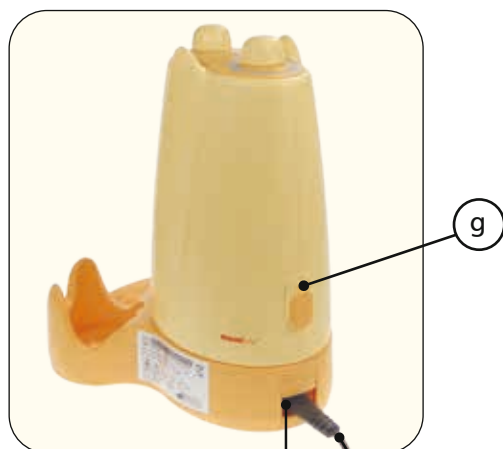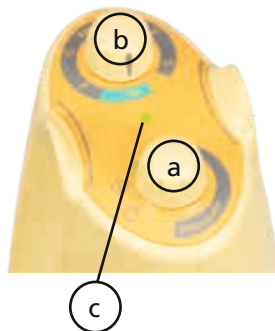


### 3. Available accessories

#### 3.1 Mobile stand

(REF 283012)

- For a fast use of the breast pump in clinical life wherever it is needed
- Mobile, stable base with five legs and five lockable castors
- Height of the mobile stand can be extended from 70 cm to 118 cm.
- For a safe transportation of the device with the mobile stand make sure that the extension is completely retracted.

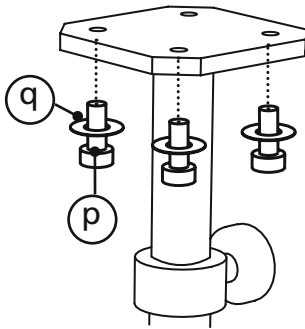#### 3.2 Components of the mobile stand

- |    |                    |                |
|----|--------------------|----------------|
| o) | Mobile stand       | (REF 283012)   |
| p) | Cylinder head bolt | (REF 49020841) |
| q) | Washer             | (REF 49026944) |
|    | Allen key          | (REF 49020001) |

#### 3.3 Assembling

Use the supplied four washers and the four cylinder head bolts to mount the **mamivac® SENSITIVE-CFH** to the metal plate of the clinic stand. Make sure the metal plate fits the dent in the bottom of the breast pump.

## 4. Using the breast pump


It is important that the clinic staff carefully instruct any patient in the proper use of the breast pump.

### Prior to using the pump, observe the following:

- Pay special attention to cleanliness. Prior to expressing, always wash your hands thoroughly with soap and wipe the breast with a damp cloth.
- To avoid malfunction and ensure a proper functioning, use sterile single-use suction kits only with original mamivac® protectors or only original mamivac® suction kits.
- Assemble the parts as shown in the operating instructions. See Chapter 5 for notes on cleaning and disinfecting.
- Please use air tubes with the following dimensions only: 5.5 mm outside Ø, 2.5 mm inside Ø, about 1 m long.

### 4.1 The mamivac® 2-phase technology

The **mamivac® SENSITIVE-CFH** breast pump simulates the baby's natural sucking behaviour. It can be divided into two phases (stimulation phase and expressing phase).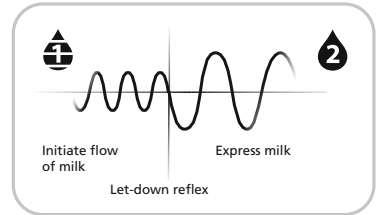


### 4.2 The stimulation phase

#### 100 cycles; fast cycles, low vacuum

Imitates the baby's sucking which is fast, but weak in the beginning. The short cycles stimulate the flow of milk.

- Turn on the pump by switching the vacuum control to . Make sure that the vacuum control indicator is in the area marked green (low vacuum).
- Turn the cycle control to a high cycle number of about 100 cycles (see figure). Make sure that the cycle control indicator is in the area marked green (fast cycles).
- When the milk starts to flow, after two minutes at the latest, switch to the expressing phase, (20-60 cycles; slow cycle, higher vacuum). A beep will sound after 2 minutes to remind you.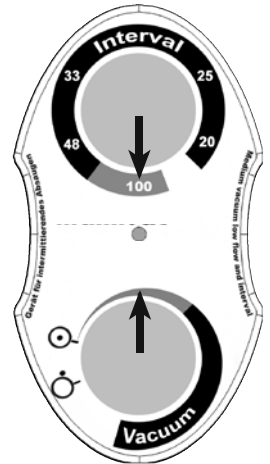


## 4.3 The expressing phase

### 20-60 cycles; slow cycles, higher vacuum

When the flow has been stimulated, the milk starts to flow. Now you can express the milk very gently and efficiently.

- Turn the cycle control indicator to the area marked grey (slower cycles).
- Turn the vacuum control indicator to the area marked grey (higher vacuum).
- In the expressing phase, both the cycle speed and the vacuum strength can be continuously adjusted.
- In general, it is recommended to reduce the cycle speed when increasing the vacuum.

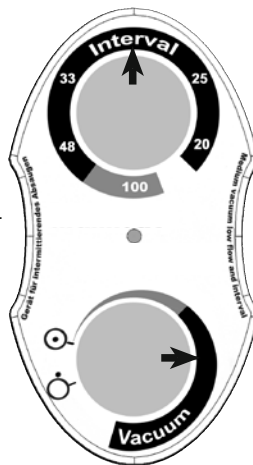## 4.4 Switching off the device

Turn the vacuum control indicator to . The device is now switched off.

## 5. Hygiene and cleaning

### 5.1 Cleaning and servicing the breast pump

- When the pump is not in use or when it is cleaned, always disconnect it from mains supply. In case of problems, also disconnect the pump immediately by all means.
- Do not use aggressive detergents (solvents, etc.) to clean the casing, but a cloth damped with a light soap sud.
- Make sure that no liquid enters the device. Wipe over the parts with a dry cloth. For a disinfection of the casing, you can use alcohol-based disinfectants.

**Do not rinse the pump!**

### 5.2 Cleaning and servicing the mamivac® suction kit

- Please observe the operating instructions of the suction kit.
- Do not use chemical disinfection baths, solvents or abrasives.
- All parts of the **mamivac® suction kits**, except for the air tube and the tube connection, can be cleaned by hand with commercial washing-up liquids or in the dish washer or boiled out in a pot at 100°C for 5 minutes.
- Prior to using the kit the next time, clean the parts which came into contact with milk. With the **mamivac® suction kits**, it is not necessary to clean the air tube or tube connection. It is sufficient to wipe off these parts with a damp cloth.
- After cleaning, rinse all parts thoroughly with lukewarm water to remove residues of the washing-up liquid, especially on those components which come into direct contact with breast milk. You might use a brush to clean areas which are difficult to reach.

**Improper cleaning and/or disinfection may lead to contamination with microorganisms and/or their toxins.**

## 6. Important safety notes

- Make sure that the supplied voltage matches the voltage stated on the power unit or is within the limits.
- Do not operate the **mamivac® SENSITIVE-CFH** breast pump in operating rooms or in rooms with explosion risk. The breast pump is not designed to be used outdoors or during transportation.
- The breast pump is intended for clinical use.

- Operate and store the breast pump in dry rooms only.
- Operate the **mamivac® SENSITIVE-CFH** breast pump only on a flat and stable surface.
- In case of disturbances or failure please contact your rental station/vendor.
- If you think a safe operating is no longer ensured, e.g. in case of a faulty insulation of the power cord or faulty cord between the power unit and the device, discontinue the operation of the device immediately (unplug power cord from wall outlet) and secure it against putting into operation again. Unplugging is the only means to safely disconnect the device from the mains supply.
- Any modifications/manipulations of the device or its accessories, or a combination with other devices and/or systems or components make a safe use impossible and are therefore strictly forbidden.
- Small parts are enclosed to the product which could be inhaled or swallowed. Keep them away from toddlers/children.
- Please check the product for any external damage and proper functioning before each use. The pump must not be used again once visible damage has been detected.
- If you experience pain while expressing or are only able to express a small amount of milk, please contact your midwife/lactation consultant or another healthcare professional.
- Any serious incidents that have occurred in connection with the product must be reported to the manufacturer and the competent national authority.

## 6.1 Note on electromagnetic compatibility (EMC)

The **mamivac® SENSITIVE-CFH** breast pump described in these instructions for use may only be operated in accordance with the following information, provided that the accessories and spare parts mentioned in chapters 12 and 15 are used, in an electromagnetic environment that corresponds to that of a domestic environment (dwelling, medical practices) where the public power system is available. It is also possible to use the **mamivac® SENSITIVE-CFH** breast pump professional health care facilities (hospitals, birth centres, etc.). It is not permitted to operate the **mamivac® SENSITIVE-CFH** breast pump in means of transportation, be it over land, across water or through the air, nor in special-use HF shielded rooms (e.g. MRT) where electromagnetic bursts of high intensity may occur nor in the immediate vicinity of active HF surgery devices or shortwave therapy devices.

A detailed technical description of the electromagnetic environment can be found in the enclosed EMC guidelines.

Provided that the **mamivac® SENSITIVE-CFH** breast pump is operated in the specified environment, an impairment of the performance characteristics given in chapter 12 or the basic safety is not to be expected.

**WARNING:** Avoid using this device directly next to other devices or with other devices stapled on top of each other as this could disturb operation.

**WARNING:** The use of accessories, converters or cables other than those defined or provided by Kimetec for this device can lead to an increased electromagnetic interference or a reduced electromagnetic immunity of the device resulting in faulty operation.

**WARNING:** Portable HF communication devices (transmitters), including their accessories such as antenna cables and external antennas, shall not be used within a distance shorter than 12 inches (30 cm) to the parts and cables listed by Kimetec for the **mamivac® SENSITIVE-CFH** breast pump. Non-observance may impair the performance characteristics of the device.

## 7. Storage, transportation and ambient conditions

- Avoid strokes or impacts of any kind, regardless of whether the device is packed or not.
- Avoid temperature changes, high air humidity or a dusty environment.
- The device can be operated at an ambient temperature of +10 °C to +40 °C, a relative humidity of 30 % to 75 % (non-condensing) and an air pressure of 700 hPa to 1060 hPa.
- Store and transport the device at an ambient temperature of -29 °C to +60 °C, a relative humidity of 85 % (non-condensing) and an air pressure of 500 hPa to 1060 hPa.
- An excessive accumulation of lint and dust in the inside of the device impairs heat dissipation of components, which may overheat as a result representing a fire hazard.

## 8. Warranty

The warranty period for the **mamivac® SENSITIVE-CFH** breast pump is 36 months from the date of purchase. Improper use, opening the device or using suction kits not specified for this breast pump will void any warranty claims.

## 9. Practical tips

Prior to using the pump for the first time and any time afterwards, clean your **mamivac® suction kit** as described in the respective operating instructions. Make sure to assemble the parts properly again as shown in the instructions. Prior to expressing, always wash your hands thoroughly with soap and wipe the breast with a damp cloth. To avoid dry, sore or cracked nipples, do not use soap or alcohol as detergent. Prepare yourself for the expressing with ease. Take a comfortable and relaxed seat.

- Gently press the breast shield over the nipple for a tight contact.
- You can operate the **mamivac® SENSITIVE-CFH** breast pump either with one or two suction kits. To save time, it is possible to express both breasts simultaneously (double pumping).
- Start to express milk. Please observe the instructions for operating explained in Chapter 4.
- The pump features a stimulation mode to support milk production. You can also use your free hand for an additional breast massage to support the flow of milk.
- Express both breasts alternately, unless instructed otherwise by your physician.
- When expressing make sure not to overfill the milk bottle. With a capacity of 125 ml, we recommend to interrupt expressing when the 100 ml mark is reached and fill the expressed breast milk to another container/bottle. Turn the bottle counterclockwise to unscrew it from the breast shield.
- Since expressing times strongly depend on the individual mother, you are asked to contact your midwife or breastfeeding assistant. As a rule of thumb, the following expressing times are recommended:

### With one suction kit:

Expressing scheme: Express first and second breast 5 minutes each one after the other, followed by 3 minutes and 2 minutes each.

### With two suction kits:

Express both breasts simultaneously for 10 minutes (double pumping). Have a look at the fill level of the bottle at the suction kit from time to time and make sure it does not overflow.

## 10. Troubleshooting

The hygienic diaphragm of the suction kit/protector effectively prevents that liquids or solids are sucked into the pump. If substances are sucked in nevertheless or any other problem occurs, immediately put the device out of operation. Contact your rental station or directly the manufacturer's customer service.

Problem	Error/solution
Pump does not start	<ul style="list-style-type: none"><li>• Use a fully charged battery or change to mains operation</li><li>• Check connections between power supply and device</li><li>• When mains operated, the LED (e) must light up in green</li><li>• Check connections between rechargeable battery and device</li><li>• When operated via rechargeable battery, the LED (f) must light up in green</li><li>• LED (f) flashes in red (error message) when rechargeable battery is weak.</li><li>• Switch off the device and remove the rechargeable battery.</li><li>• If you cannot solve the problem, return the device.</li></ul>
Pump does not suck (does not build up vacuum):	<ul style="list-style-type: none"><li>• Status LED (c) lights up in red (error message)</li><li>• Try to switch off and on the device several times. Wait a minute before switching on again. After five unsuccessful attempts, return the device to the manufacturer.</li><li>• Check the suction kit:</li><li>• Check breast shield valve (for cracks and soiling)</li><li>• Check protective diaphragm for cracks</li><li>• Protective diaphragm must be inserted flush and crinkle free</li><li>• Check tube connections</li></ul>

The device is maintenance-free. For repair work, it must be sent to the manufacturer. The intended function can be checked by the manufacturer or by means of the test unit for mamivac® breast pumps (REF 2899120). Regular testing of electrical safety (recommended at intervals of 2 years) can also be performed by the manufacturer or another authorized person who has the required expertise and measuring and testing equipment.

The breast pump/power unit must not be opened by users/patients.

## 11. Environmental protection and disposal

Please dispose of **mamivac® SENSITIVE-CFH** breast pumps out of order in accordance with your local disposal regulations for electrical devices.

## 12. Technical data

<b>mamivac® SENSITIVE-CFH:</b>	<b>REF 286000</b>	<b>Device classification</b>
Nominal voltage:	100-240 V/AC 50 - 60 Hz	Classified acc. to 93/42/EEC
	max. 0.62 A	Article 9, Appendix IX: Class IIa/Rule 11
H x W x D:	24 cm x 21 cm x 22 cm	
Weight:	1.3 kg	
Max. vacuum:	-5 to -33 kPa (±15%)	
Mean vacuum:	Device for intermittent sucking without continuous vacuum	
Power cord:	2 cores 2 x 0.75 mm; H05VVH2-F 3.0 m long with Euro plug (REF 283003)	
Protection class:	II	

## 13. Conversion table of units

<b>1 kPa</b>	corresponds:	0.01 bar
<b>0.14504 PSI</b>	corresponds:	7.5006 mmHg
(pounds per square inch)		(millimetre mercury, at 0°C)

## 14. Legend

	Observe safety instructions		Manufacturer
	Observe operating instructions		Protect from sunlight
	Store in dry place	<b>IP40</b>	Protection against penetration of solid particles of a diameter $\geq 1.0$ mm. No protection against penetration of water.
	Conformity mark according to EU Council Directive 93/42/EEC		Direct current
	Device of protection class II		Alternating current
	Patient applied part of type BF		Do not dispose in domestic waste
	On	<b>SN</b>	Serial number
	Off	<b>REF</b>	Item number
	Observe operating instructions		

## 15. Accessories and spare parts

Designation	REF
mamivac® SENSITIVE-CFH breast pump (device)	281021
mamivac® SENSITIVE-CFH with country-specific power cord (K*)	286000K*
Test unit	2899120
Clinic stand	283012
mamivac® suction kit ECONOMY L	283122
mamivac® double kit ECONOMY L	283124
mamivac® suction kit PREMIUM L	283120
mamivac® double kit PREMIUM L	283123
Breast tube XL for suction kit PREMIUM L	283121
Power cord EU	283003
Power cord UK	287001
Power cord US	287002
Power cord AU	287003
Operating instructions mamivac® SENSITIVE-CFH	GB1010029M

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